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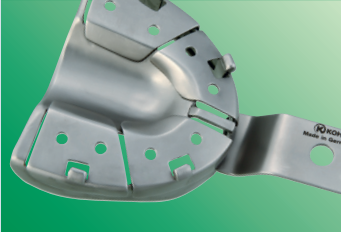
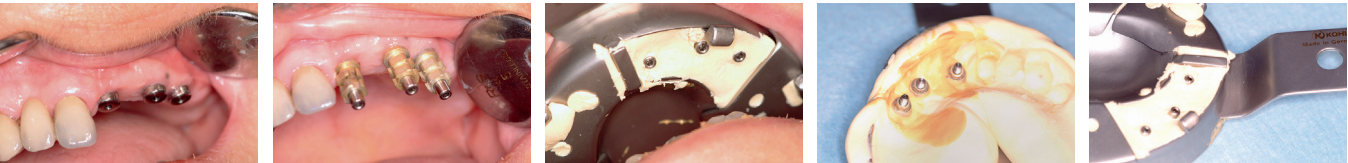
The segments (now only 4 segments) of the WINTRAY® NEW GENERATION are easily removed by a simple click-system. No screws, no screwdriver anymore!

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Doctor Bernard Guillaume
MaxilloFacial surgeon and Implantologist
Founder President of the French College of Implantology
Member of the French College of Stomatology and Maxillofacial Surgery
Member of the Syndicate of the Scientific Press
University degree: Biomaterial and Tissue calcified, dental implantology
Expert near the Court of Appeal of Paris and the Administrative Court of Paris

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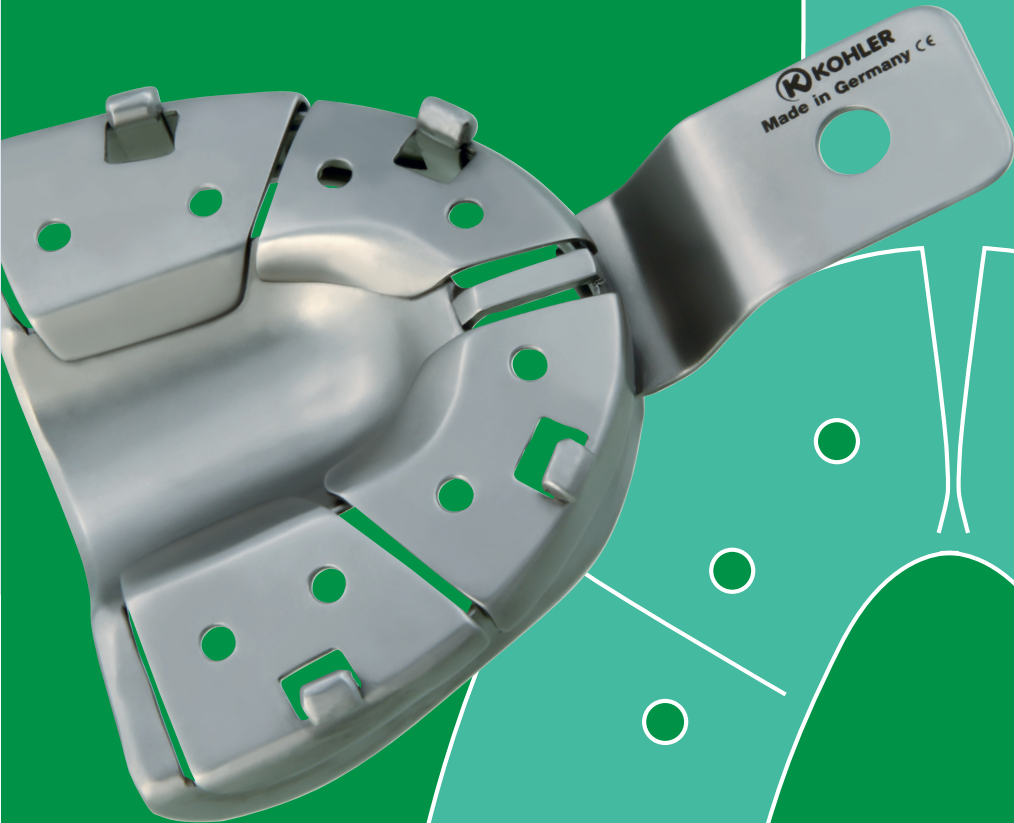
Clinical Report:
Impression taking during fabrication of implant-supported restorations

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INTRODUCTION

The WINTRAY® Impression Tray New Generation was developed by Dr. Guillaume and Dr. Mazeirat to enable taking an accurate impression of dental implants.

In implant prosthetics the prosthetic result depends directly on the reliability of the pick-up impression, which is required for fabricating the laboratory working model. Stock trays or custom trays made from resin materials do not meet the biomechanical requirements of implant restorations. In order to take pick-up impressions of dental implants for the dental laboratory, dentists must currently use a stock tray and drill some holes through it, therefore destroying the tray, or get a perforated custom tray fabricated in a dental laboratory at considerable costs

NEW CONCEPT

The WINTRAY® New Generation was developed in a way that several segments can be removed at the position of the implant.

It is much easier and faster with the latest generation of WINTRAY®, as the segments are only retained using a simple retentive clip instead of a screw compared to our previous well-established model.

The low tray walls also allow closer proximity to the alveolar ridge to produce greater emergence of the impression posts, which is then more clearly visible intraorally. In addition, the shorter tray walls do not press as strongly on the mucosa and are therefore less painful for patients with edentulous jaws. Due to the fact that the windows are larger now the new WINTRAY® can be used of pick-up impressions with divergent axes, which is very useful in the case of superstructures on several abutments.

INSTRUCTIONS FOR USE

The application of the new WINTRAY® is now even easier:

1. Select the size of impression tray and remove the appropriate segments for the posts in the dental arch from the impression tray.
2. Load the impression tray with impression material and press it onto the dentition until the screws in the impression posts can be felt.
3. Once the material has hardened, loosen the holding screws of the impression posts and remove the impression tray from mouth. The impression posts remains firmly positioned into the impression.

Do not use adhesives!

PLACING IMPLANTS IN EDENTULOUS CASES

When fabricating implant-supported full dentures, several to all segments can be removed. The impression tray is used in the same way.

CLEANING AND STERILIZATION

It is made of stainless steel and as a consequence sterilisable and fulfils therefore all requirements on the traceability of the procedure of the sterilisation.

- Clean the impression tray and the removed segments carefully by using customary cleaning solutions.

- Click in all removed segments again by inserting at first the brad into the drilling. Attention please take care of the correct position of the segments!

- Take through the sterilization procedure in the normal way (max. 200°C/389°F).

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