# WINTRAY®

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## Introduction

The WINTRAY® impression tray was developed with Dr. Guillaume and Dr. Mazeirat to enable taking an accurate impression of dental implants.

In implant prosthetics the prosthetic result depends directly on the reliability of the pick-up impression, which is required for fabricating the laboratory working model. Stock trays or custom trays made from resin materials do not meet the biomechanical requirements of implant restorations. In order to take pick-up impressions of dental implants for the dental laboratory, dentists must currently use a stock tray and drill some holes through it (destroying the torsional stiffness) or get a perforated custom tray fabricated in a dental laboratory at considerable costs.

## Concept

The WINTRAY® was developed in a way that several segments can be removed at the position of the implant.

The segments of WINTRAY® GENERATION I are unscrewed, the ones of GENERATION II are clipped out the tray.

The WINTRAY® GENERATION I has the height of a regular standard tray with RIM-LOCK. The GENERATION II allows due to its lower tray walls a closer proximity to the alveolar ridge to produce greater emergence of the impression posts, which is then more clearly visible intraorally. In addition, the shorter tray walls do not press as strongly on the mucosa and are therefore less painful for patients with edentulous jaws. Due to the fact that the windows are larger the new WINTRAY® can be used of pick-up impressions with divergent axes, which is very useful in the case of superstructures on several abutments.

## Instructions for use

The application of both versions is the same:

1. Select the size of impression tray and remove the appropriate segments for the posts in the dental arch from the impression tray.
2. Load the impression tray with impression material and press it onto the dentition until the screws in the impression posts can be felt.
3. Once the material has hardened, loosen the holding screws of the impression posts and remove the impression tray from mouth. The impression posts remains firmly positioned into the impression.

Do not use adhesives!

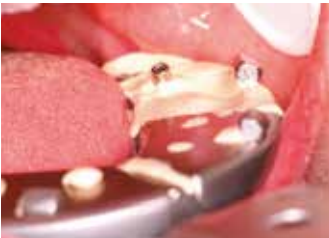
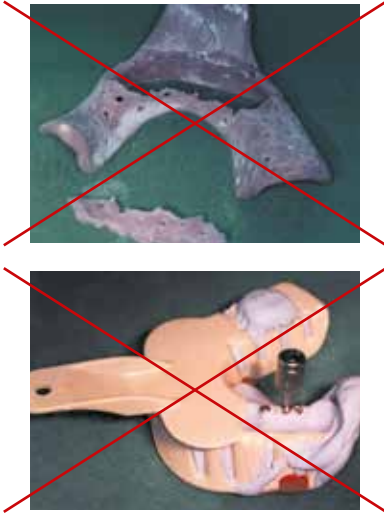
## Placing implants in edentulous cases

When fabricating implant-supported full dentures, several to all segments can be removed. The impression tray is used in the same way.

## Cleaning and Sterilization

It is made of stainless steel and as a consequence sterilizable and fulfils therefore all requirements on the retraceability of the procedure of the sterilization.

- Clean the impression tray and the removed segments carefully by using customary cleaning solutions.
- Mount all removed segments again. The segments of GENERATION I are screwed, the ones of GENERATION II are clipped (pay attention first to insert the pin into the drilling, then with slight pressure push the clip spring through the opening. Take care of the correct position of the spring and the opening).
- Take through the sterilization procedure in the normal way.

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**WINTRAY® GENERATION I**

- 9 removable segments
- especially suitable for the Pick-up impression technique
- constructed from Stainless Steel
- re-usable – can be sterilized
- extremely rigid for high precision impressions
- perforated for retention of impression material

All WINTRAY® are delivered including 4 screws and 1 key.

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The segments (only 4 segments) of the WINTRAY® GENERATION II are easily removed by a simple click-system. No screws, no screwdriver anymore! Due to its reduced height the WINTRAY® is particularly suited for the impression in edentulous jaw.

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**Kohdent Roland Kohler Medizintechnik GmbH & Co. KG**