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**Clinical report:**

Impression taking during fabrication  
of implant-supported restorations

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**GB** The success of implant-supported restora- tions depends on taking an accurate impression of the impression post and using it when casting the working model. Neither stock nor custom plastic impression trays meet up to the bio- mechanical demands of implants. In this article, we would like to present a new, patented pick-up impression system: The WINTRAY® is an impression tray with removable segments – it is rigid, re-usable, can be sterilized and only a single appointment is required for casting a perfect working model.

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### INTRODUCTION

Impression materials combine highly precise repro- duction of details with perfect three dimensional stability, provided that they are retained properly in the impression tray and cannot distort (rigid impression tray). The pick-up procedure is considered the most precise method for positioning the implant analogues on the working model. Actually, the impression posts are retained in the impression material so firmly that once disconnected from the implants they are not stressed when the impression tray is removed from the mouth, even if the implants are not parallel. The pick-up procedure is recommended by most major implant manufacturers.

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To date, the procedure involved two appointments as follows:

- During the first session the pick-up impression post was placed and an alginate impression taken for fabricating an acrylic custom impression tray (Figure 2, impression tray broken accidentally by hand)
- During the second session, the pick-up impression post was replaced and the custom impression tray tried in – it required trimming several times.
- The impression tray was loaded with elastomeric impression material and the final impression taken.

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A stock or custom plastic tray will be weakened by the perforations and openings for the pick-up screws and certainly would not withstand this rigorous bio- mechanical implant procedure.

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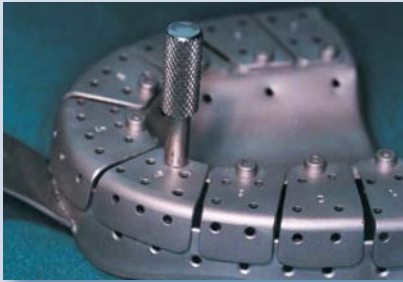
## GB NEW CONCEPT

This is a new type of tray: The WINTRAY® fulfils all the criteria for taking impressions for implant-supported restorations.

The inside of the impression tray with removable segments. Please note the perforations and gaps for retaining the elastomer.

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This impression tray for implants is made of surgical stainless steel – it includes segments on its occlusal areas which can be removed with the screws. The segments are L-shaped – the longest parts fit the occlusal areas of the tray and the short parts fit the vestibular areas; the vestibular openings are used in case of centripetal bone resorption where the implant axes must be angled toward the vestibular aspect.

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The impression tray in position.  
Please note the buccal opening.

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This extremely rigid impression tray includes numerous retainers (perforations in the tray itself and removable segments, gaps between the segments and tray, the peripheries, indentations and screw heads.)

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## GB INSTRUCTIONS FOR USE

The procedure is very straightforward:

- The appropriate segment for the pick-up post in the dental arch is removed from the impression tray.
- The impression tray is loaded with impression material and pressed onto the dentition until the screws in the impression posts can be felt.
- Once the elastomer has hardened, the screws are loosened and the impression tray removed from the mouth.

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Taking an impression of a maxillary implant

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Taking an impression of a mandibular implant

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The impression in the mouth and loosening the screw

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**GB** The abutment on the implant

Fertiger Zahnersatz in situ

The finished restoration in the mouth

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The impression tray is sterilized using the same concept as for surgical materials: When sterilized with the screwdriver, the procedure is guaranteed to be traceable thus complying with the requirements of current standards.

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### INDICATIONS

#### Single or multiple implant placement

The WINTRAY® with removable segments can be used for taking impressions of all sectors of the dental arches, regardless of their shape, as it is available in several sizes. The correct numbered segment for the clinical situation is removed to expose the implant zone precisely.

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Taking the impression. Please note the screw protruding from the impression tray.

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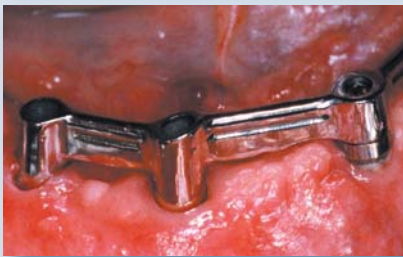
#### Placing implants in edentulous cases

When fabricating implant-supported full dentures several segments can be removed. The impression tray is used in a similar manner.

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#### Tooth-borne restorations

As the impression material is accessible, when fabricating tooth-borne restorations the impression can be removed from the model without fracturing the slender, fragile dies.

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#### CONCLUSION

The WINTRAY® is a re-usable impression tray with removable segments which saves a great deal of time for the dentist and patient as well as producing outstanding impressions. It enhances the standard of our treatment and reduces costs.

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Medizintechnik GmbH & Co. KG