waveline®

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GB

WAVELINE® Osteotomy instruments are multi-functional and enable users to widen their range of oral surgery considerably. The instruments developed by Dr. F. A. Poulmaire provide for a wide range of ergonomic advantages and optimize the handling which enhances the quality of the outcome.

The use of the WAVELINE® instruments aims at:

- increasing the density of poorly mineralised and relatively malleable bone (bone grade D3 and D4) via expansion and condensation, so that reliable results which remain stable long term can be achieved.
- splintering the bone to augment the bony alveolar ridge.
- penetrating the sinus floor in a controlled manner and raising the mucous membrane of the nose, so that adequate amounts of bone augmentation material can be placed to ensure afterwards that the implant is retained firmly.

In comparison to conventional systems the WAVELINE® instruments offer some specific advantages:

(1) WAVELINE® handle

(a) Because of the special WAVELINE® handle different stop positions are possible. As the fingers find a perfect halt in the borehole a twisting of the instrument during use is almost impossible.

(b) Absorption of the blow

Due to the waved handle and to the fact that the user positioned the fingers on one or several depressions, the patients feel the hammer blows during the treatment less intensively than by round and massive instruments. The shock waves will be absorbed by the fingers and the hand. When using the Summers technics 73 % of the patients assessed the treatment with WAVELINE® instruments as more pleasant than using massive instruments with round handle. During the condensation and expansion 61 % of the patients prefer a treatment with WAVELINE® instruments.*)

(c) Trapped instrument

Trapped instruments can be freed easily by using a second instrument with a careful rotation.

(2) Coloured depth markings

The depth markings are affixed in different colours. The user memorizes only the colour of the maximal working depth. A counting of the grooves is not necessary.

(3) WAVELINE® hammer

The WAVELINE® hammer combines the efficacy and the aesthetics of the WAVELINE® Osteotomy instrument set. The position of the fingers is defined clearly. The hammer head can be used either with its metal or silicone end.

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Pin-Körner und Pin-Osteotome

Pin center-punches and pin-osteotomes

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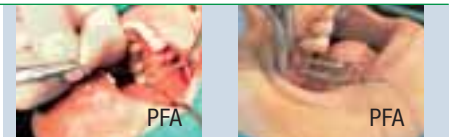
GB The WAVELINE® directional gauges have the same property profile as the pin center-punches No. 7381. They indicate implant alignment at the prosthetic restoration level and allow a three dimensional harmonisation of the planned implants with each other. Furthermore it ensures that the so-called memory effect of the bone tissue can be deleted and that the tissue will not resume to its previous position. The directional gauges are appointed with a security tag.

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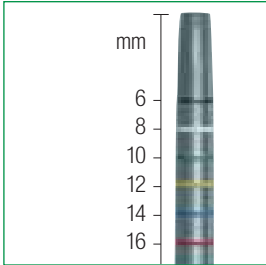
GB

WAVELINE® osteotomes

The WAVELINE® osteotomes are very concave. On the one hand, this allows the bony surfaces to be „scraped“ more effectively. On the other hand, larger amounts of bone chips can be harvested and carried.

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Progressive order

The WAVELINE® osteotomes are cylindrical-conical and build a progressive order. Due to their diameters they are neutral to the used implant system.

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The specific characteristics of the WAVELINE® bone spreaders are the differences in the shape of spreading instruments, which show both a vestibular and a palatal surface.

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The vestibular surface is frequently prone to cracking and if pressure is exerted by cylindrical-conical osteotomes it may even cause fractures.

The WAVELINE® bone spreaders contribute twice to preserving the vestibular alveolar ridge:

- (1) Due to the shape of a slightly convex chisel the force is exerted on a larger surface area.
- (2) Because of the specifically shaped tip the force on the palatal surfaces will be much heavier than on the vestibular surface.

To assure that the instruments always affect the vestibular surface with a smaller force only the vestibular surface has the graduation of the depth and the colour coding.

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Medizintechnik GmbH & Co. KG